



# EXray<sup>®</sup> 20



Snap-off  
Threaded pin



SynchroMedical

*The interphalangeal reference*

# EXray<sup>®</sup> 20 PEEK

## Snap-off threaded pin

### DESCRIPTION

EXRAY<sup>®</sup> 20 snap-off threaded pin has been developed as a resectable peek pin. The seat of the implant is prepared with a 2 mm drill. The pin consists of a trocar tip, a threaded portion and a portion with a narrower diameter that can be connected to a power tool.

The EXRAY<sup>®</sup> 20 has a non-compressive  $\varnothing$ 2.5mm continuous thread. The 3 markings are positioned at 15 mm, 20 mm and 25 mm. The full length of the thread is 30 mm. Once implanted, the excess length is removed with a saw blade.

EXRAY<sup>®</sup> 20 snap-off threaded pins are manufactured in PEEK as per standard ASTM F2026.

- Simple preparation
- Variable screw length
- Used to support stabilization of osteotomies
- Simplified revision
- PEEK threaded pins are radiolucent
- The elasticity modulus is close to the one of bone



### ORDERING INFORMATION

#### Implant

PW2060.....L 60 mm



#### Instruments

	AO20ST100	DRILL
	PW9010	PROTECTIVE BARREL
	HV9016	SCREW EXTRACTOR

*The counterclock screw extractor (Ref. HV9016) is used for revision*

All rights reserved © Pictures non-contractual